

	Document No.: YSCE-LJG-02-02	Version: 04	Page: 1 / 5
CE Technical Documentation	IFU	Effective Date: 2025-02-14	

Instruction for Use



<Product: Disposable pressure extension tube >

<Document No.: YSCE-LJG-02-02 >

< Version: 04 >

Issued By		Date	
Reviewed By		Date	
Approved By		Date	

Wuxi Yushou Medical Appliances Co., Ltd.

No.115 Nongxinhe Rd, Xishan District, 214191 Wuxi, Jiangsu Province, People's Republic of China.

	Document No.: YSCE-LJG-02-02	Version: 04	Page: 2 / 5
CE Technical Documentation	IFU	Effective Date: 2025-02-14	

Instruction for Use

Disposable pressure extension tube

[Model specifications]

Disposable pressure extension tubes are offered in four distinct models: the single tube type, the single coiled tubing type, the U tube type, and the Y tube type. Regarding the tube length, it comes in various options including 30 cm, 35 cm, 75 cm, 120 cm, 150 cm, 180 cm, 225 cm, 250 cm. For the outer diameter (OD) of the tube, there are two choices: 3.5mm and 4.0mm.

REF.	Product Specification	Type	Length (cm)	ID (mm)	OD (mm)
120700000001	YL30 (3.5pipe,30cm)	Straight	30cm	2.0	3.5
120700000002	YL35 (3.5pipe,35cm)	Straight	35cm	2.0	3.5
120700000003	YL75 (3.5pipe,75cm)	Straight	75cm	2.0	3.5
120700000004	YL90 (3.5pipe,90cm)	Straight	90cm	2.0	3.5
120700000005	YL120 (3.5pipe,120cm)	Straight	120cm	2.0	3.5
120700000006	YL150 (3.5pipe,150cm)	Straight or coiled	150cm	2.0	3.5
120700000007	YL30A (4.0pipe,30cm)	Straight	30cm	2.5	4.0
120700000008	YL35A (4.0pipe,35cm)	Straight	35cm	2.5	4.0
120700000009	YL75A (4.0pipe,75cm)	Straight	75cm	2.5	4.0
120700000010	YL90A (4.0pipe,90cm)	Straight	90cm	2.5	4.0
120700000011	YL120A (4.0pipe,120cm)	Straight	120cm	2.5	4.0
120700000012	YL150A (4.0pipe,150cm)	Straight or coiled	150cm	2.5	4.0
120700000033	YYL150 (3.5pipe,150cm)	Y type or U type	150cm	2.0	3.5
120700000034	YYL180 (3.5pipe,180cm)	Y type or U type	180cm	2.0	3.5
120700000035	YYL225 (3.5pipe,225cm)	Y type or U type	225cm	2.0	3.5
120700000036	YYL250 (3.5pipe,250cm)	Y type or U type	250cm	2.0	3.5
120700000037	YYL150A (4.0pipe,150cm)	Y type or U type	150cm	2.5	4.0
120700000038	YYL180A (4.0pipe,180cm)	Y type or U type	180cm	2.5	4.0
120700000039	YYL225A (4.0pipe,225cm)	Y type or U type	225cm	2.5	4.0
120700000040	YYL250A (4.0pipe,250cm)	Y type or U type	250cm	2.5	4.0

【Intended Use】

Disposable pressure extension tube is suitable for single-use sterile fluid lines for pressure infusion equipment with a maximum pressure of 200kPa

Intended Patient Populations】

	Document No.: YSCE-LJG-02-02	Version: 04	Page: 3 / 5
CE Technical Documentation	IFU	Effective Date: 2025-02-14	

This device should only be used by professional medical staff who have been fully trained and qualified in the use of infusion techniques.

【Intended user】

Patients who are under liquid infusion, which include adults, children and newborn.

【Intended using environment】

It is used by professionally trained physicians in medical institutions.

【Potential complications】

There are no noted complications.

【Indications】

There are no specific medical conditions to be diagnosed, treated or monitored.

【Contraindications】

No absolute contraindications when used correctly by medical personnel.

【Potential side-effects】

There is no noted side effect for this device.

【Cautions】

- The product is for single use only. Do not reuse.
- If the product is damaged or contaminated, do not use the product.
- Verify the length and type of the devices and ensure correct assembly use. The product shall be handled by health care professionals.
- Pay attention to the possible toxicity of high-risk population (such as premature, newborns, pregnant women, lactating women, prepubertal men) and choose alternatives as far as possible.

【Warning】

- The product is prohibited to infuse drugs incompatible with PVC.
- Do not use if the package is damaged or unintentionally opened before use.
- Do not re-sterilize. Do not reprocess.
- Do not use the product if the shelf life has expired.
- Use immediately after unpacking.
- Don't Store at extreme temperatures. Product should be stored under normal temperature and humidity
- Dispose of safely after use to avoid risk of infection.
- Use time should not exceed 24 hour.

Combination with other products

Disposable pressure extension tube is used for single-use sterile fluid lines for pressure infusion equipment with a maximum pressure of 200kPa.

- **Baxter International:** Product Line: Spectrum IQ Infusion System with Dose IQ Safety Software
- **BD (Becton, Dickinson and Company):** Product Line: Alaris Infusion Pumps
- **Smiths Medical:** Product Line: CADD Infusion Pumps
- **Terumo Medical Corporation:** Product Line: TE-331/332 Pressure Monitoring Kit

【Principles of operation】

- The product has been sterilized by ethylene oxide and is operated by professional medical personnel.

	Document No.: YSCE-LJG-02-02	Version: 04	Page: 4 / 5
CE Technical Documentation	IFU	Effective Date: 2025-02-14	

- The Disposable pressure extension tube is mainly to the human body infusion of pharmaceutical liquid pipeline. Please check whether the single package is broken, there is any foreign body in the package, the parts fall off before using this product. If it is found that the single package is broken, there is a foreign body in the package and the parts fall off; the product should be strictly prohibited and destroyed.
- Tear the package according to gap (or tear mark) on the package, take out the product, pull out the sheath.
- Align the interface to be connected, and then tighten it and use it;
- Put the product into an approved disposal container as soon as possible after use and dispose of the waste in accordance with relevant hospital regulations.

[Storage conditions]

No special storage or transportation conditions. Recommendations are storage in room temperature, in dry places and avoiding exposure to strong light.

[Disposal]

The product is disposable and should be disposed of according to local medical waste regulations after use.

[Shelf-life]

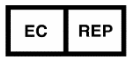
5 years. Use time should not exceed 24 hour.

[Sterilization]

The product is sterilized by EO. If the package is damaged, please do not use it

The user and/or patient shall report any serious incident that has occurred in relation to the device to Wuxi Yushou Medical Appliances Co., Ltd and the competent authority of the EU Member State in which the user and/or patient is established.

[Definitions of symbol used]

Signs	Description	Signs	Description	Signs	Description
	Trademark of Manufacturer		Batch number		Date of Manufacture
	Medical device		Model number		Country of manufacturer
	Use-by date		Sterilized using ethylene oxide		Caution
	Non-pyrogenic		Catalogue number		Unique Device Identifier
	Single sterile barrier system		Double sterile barrier system		Keep away from sunlight
	Do not reuse		Do not use if the package is damaged		Keep dry
	Do not resterilize		Authorized representative in the European		Manufacturer

	Document No.: YSCE-LJG-02-02	Version: 04	Page: 5 / 5
CE Technical Documentation	IFU	Effective Date: 2025-02-14	

Signs	Description	Signs	Description	Signs	Description
			Community		
	CE marking of conformity, and Notified Body Code		Consult instructions for use	/	/

[Information of the Manufacturer]

Name: Wuxi Yushou Medical Appliances Co., Ltd.

Address: No.115 Nongxinhe Rd, Xishan District, 214191 Wuxi, Jiangsu Province, People's Republic of China.

[Information of EU Authorised Representative]

The contact information of EU representatives is shown as follows:

EU representative: Shanghai International Holding Corp. GmbH (Europe)

Address: Eiffestrasse 80, 20537 Hamburg, Germany

Tel: +49-40-2513175

Fax: +49-40-255726

E-mail: shholding@hotmail.com